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## Alloy characteristics

Biocertificate	<input checked="" type="checkbox"/>	Vickers hardness (HV 5)	* 155/220/225
Type (ISO 22674)	4	BEGO-GOLD colour code	yellow 3
Density [g/cm <sup>3</sup> ]	15.5	Average grain size [µm]	25
Preheating temperature [°C]	700	Heat treatment	400 °C 15 min
Casting temperature [°C]	1050	Soft annealing	750 °C 10 min
Melting interval [°C]	900–940	(then quenching in water at 20 °C)	
Modulus of elast. approx. [GPa]	95		
Elongation limit (R <sub>p0,2</sub> ) [MPa]	* 340/500/510	* soft / after casting / hardened	
Ductile yield (A <sub>5</sub> ) [%]	* 40/15/14		

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## Safety hint

Metal dust is harmful to your health.  
When deflasking and blasting use a suction  
extraction system and breathing mask type  
FFP3-EN149:2001!

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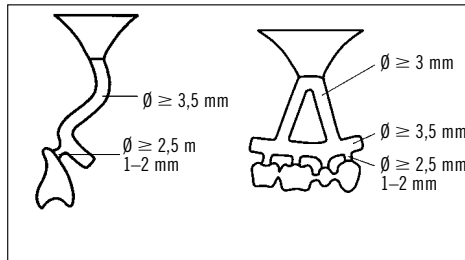
## Instructions for use

## Modelling:

- Minimum metal thickness (after grinding): for acrylic veneering with retention pearls 0.3 mm.
- Spruing of single crowns: provide casting reservoir.

**Investing:** Use phosphate-bonded or plaster-bonded crown and bridge investment materials (e.g. BellaStar, Bellavest®).

**Casting and finishing:** General: Do not overheat alloy. Use only clean crucibles, one crucible per alloy. Recommendation: to enable an exact identification of each case cast new metal only.

- In case of re-casting: only re-cast identical alloys. Blast old material. Add at least 50 % of new material.
- Use ceramic or graphite crucible. Ceramic crucible: sprinkle some Auromelt HF melting powder on the casting ingots.
- Continue to heat after complete melting of the casting pieces:  
Flame melting, immediate casting!  
HF induction heating, immediate casting!  
Resistance heating 2–3 min

After deflasking, blast residual investment material with Korox® 25, Korox® 50 or Korox® 110 at 2–3 bar. Use fine carbide, ceramically bonded stones or BEGO sintered diamond milling tools for finishing.

**Acrylic veneering:** To simplify rubber polishing, blast-polish with Perlablast® micro (lead-free soda glass). Then rubber polish with BEGO's green rubber polishers for precious metal and polish with BEGO paste for preliminary and subsequent polishing of precious metals. The relevant instructions must be followed for processing the acrylic veneering materials.

## Soldering and polishing:

- Support object in a soldering block of Bellatherm®. Prepare a gap of max. 0.2 mm with parallel walls.
- Soldering with:  
BEGO-Gold Solder I (810 °C) (REF 61017) / Minoxid flux,  
BEGO-Gold Solder II (765 °C) (REF 61043) / Minoxid flux or  
Pre-flux U Gold Solder (810 °C) (REF 61028) with pressed-in flux.
- Acid-treat residual flux and metal oxides (60 °C, 20–30 seconds). Clean thoroughly (steam clean or boil in aqua dest.).
- Polishing is possible in commercially available gold polishing units.

**Laser welding:** Filling material: PlatinLloyd® 100 wire, dia. 0.35 mm (REF 61152) or cast one yourself.

Prescription only / For professional use only 

**Secondary effects:** Such as allergies to contents of the alloy or electrochemically based reactions may very rarely occur.

**Reciprocal actions:** In case of occlusal or approximal contact of different alloys electrochemically based reactions may very rarely occur.

**Reactions:** In case of known incompatibilities and allergies to contents of the alloy.

**Warranty:** Whether given verbally, in writing or by practical instructions, our recommendations for use are based upon our own experience and trials and can only be considered as standard values. Our products undergo constant further development and are therefore subject to modification regarding design and composition.